




Reliable Fixation of  
GTR/GBR Membranes

# Ti-System

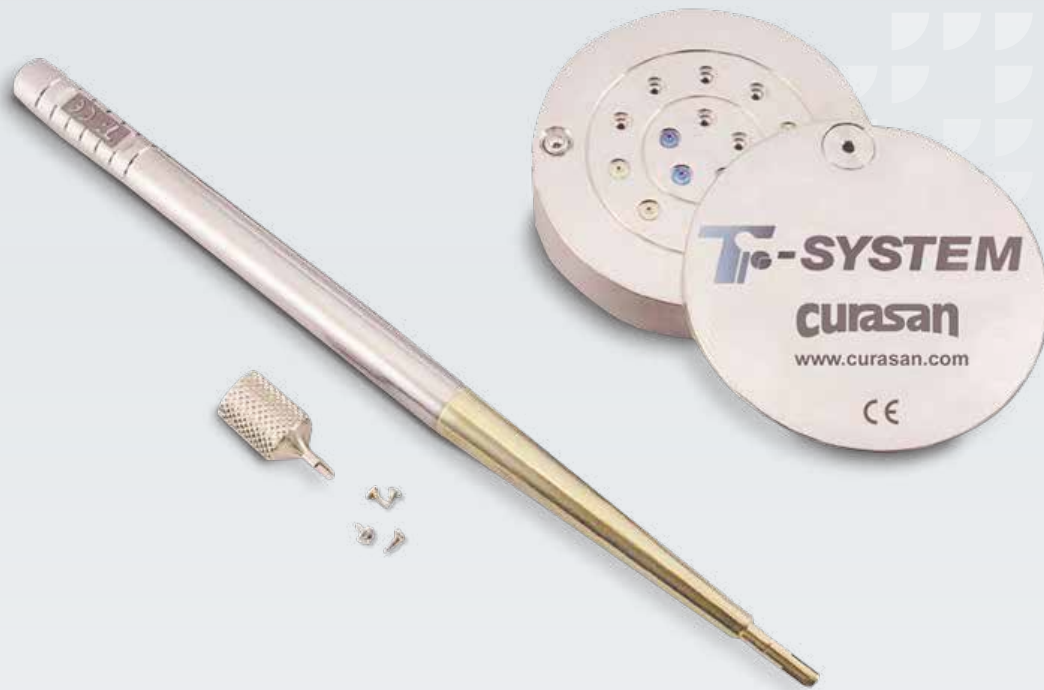

1

## Reliable Stability

- Safe, fast and simple fixation of resorbable and non-resorbable membranes<sup>1</sup>
- Effective immobilization of membrane and bone graft particulates for undisturbed healing<sup>2</sup>

2

## Titanium Tacks<sup>1</sup>

- Hex connection for secure pick up and stable transportation to the surgical site
- Made of Titanium Grade 5
- Available in 3 mm length in pack of 5 units

3

## Innovative Tack Design "Nail In – Screw Out"

- Threaded design for rigid fixation and retention
- Easy to remove with the hex screw driver<sup>1</sup>

4

## Autoclavable Tray

- Space for 15 titanium tacks
- Easy to open thanks to one-hand mechanism
- For machine cleaning and sterilization<sup>1</sup>

1. Ti-System IFU latest revision. November 1st 2014

2. Fugazzotto PA, Implant and regenerative therapy in dentistry: a guide to decision making. 2009: John Wiley & Sons.

# Ti-System

## How It Works



1  
Sterilization of all components prior to use.



2  
Pick the tack from the tray using the application device and place it in-situ using a mallet.



3  
At the day of re-entry use the hex screw driver to loosen the tack and remove it with sterile tweezers.

## Typically Used For



**Fig. A** Lateral Sinus lift procedures to fix CopiOs® Pericardium Membrane.



**Fig. B** Horizontal ridge augmentations to stabilize bone graft and CopiOs Pericardium Membrane.



**Fig. C** Vertical grafting procedures to fix Titanium reinforced OsseoGuard® PTFE Membrane.

Clinical photos. ©2020 Dr. Alejandro Prieto, Spain. All rights reserved. Individual results may vary.

## Ordering Information

Item no. #	Description	Product Class
CUR.9000810101	Titanium endpiece, straight	I
CUR.9000810103	Titanium tack handpiece	I
CUR.9000810104	Hex screw driver	I
CUR.9000810105	Sterilization tray (15 tacks max)	I
CUR.9000810106	Titanium Tack, length 3 mm - pack of 5	IIb

For more information, visit [ZimVie.com](http://ZimVie.com)

**ZimVie Global Headquarters**  
4555 Riverside Drive  
Palm Beach Gardens  
FL 33410  
Phone: +1-561-776-6700  
Fax: +1-561-776-1272  
[dentalCS@ZimVie.com](mailto:dentalCS@ZimVie.com)

ZimVie

Unless otherwise indicated, as referenced herein, all trademarks and intellectual property rights are the property of ZimVie Inc. or an affiliate; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. The CopiOs Pericardium Membranes is manufactured by Tutogen Medical GmbH. The OsseoGuard PTFE Membrane is manufactured by Osteogenics Biomedical, Inc. The Ti-System is manufactured by Curasan AG. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie Dental. ZV0596 REV A 02/23 ©2023 ZimVie. All rights reserved.

