

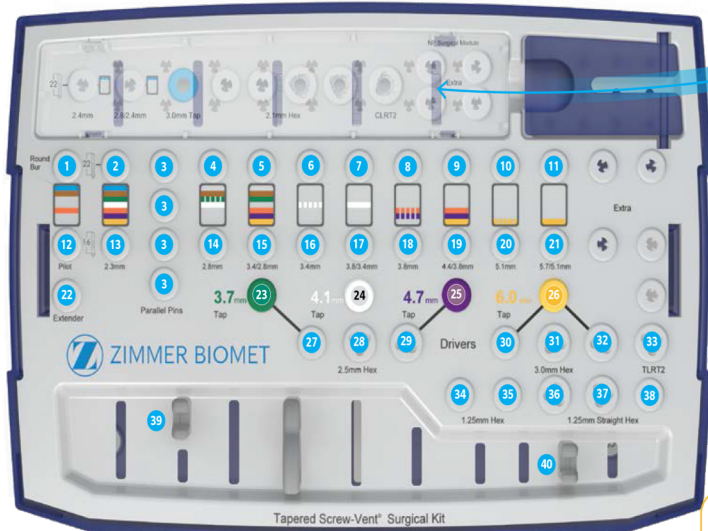



TSV Surgical Kit Layout Chart for Tapered Screw-Vent, Trabecular Metal & Eztetic Implant Placement Drive Gold Drills

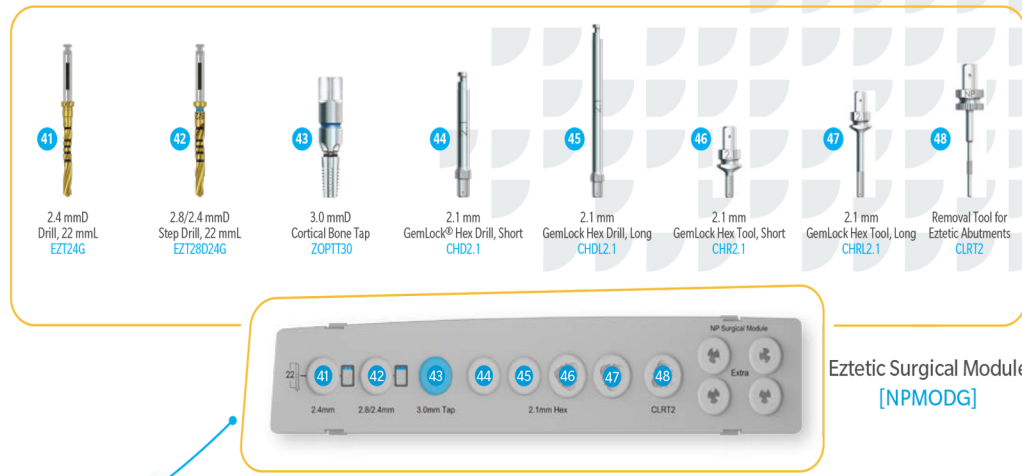

TSVKITG

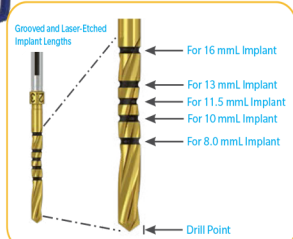

Eztetic Surgical Module [NPMODG]

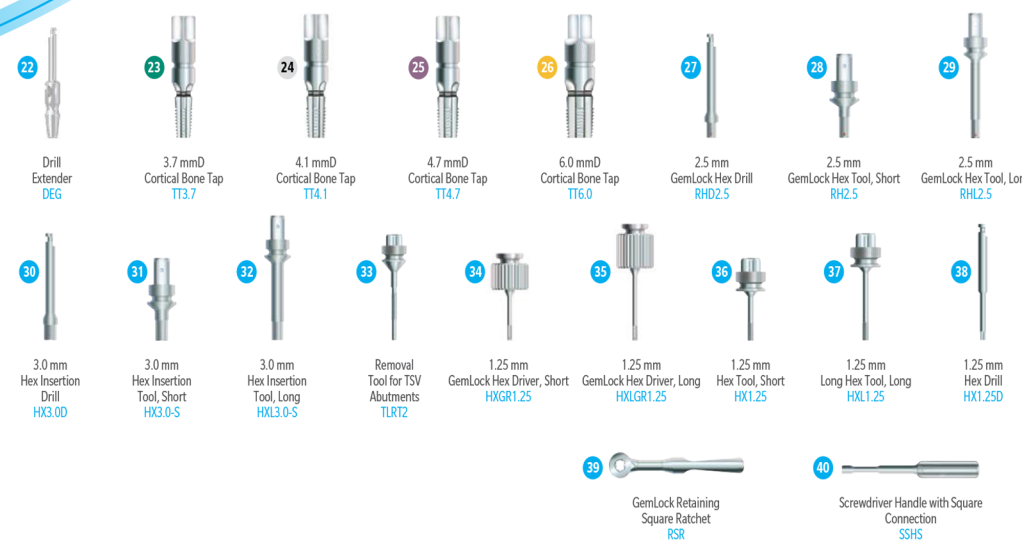

Drilling Sequence Guidelines

Soft-Bone Protocol: follow solid color bars on the surgical tray surface until the segmented color bar. The segmented color bar indicates the final drill for soft-bone protocol.

Dense-Bone Protocol: Follow solid color bars only. The last solid bar in the sequence represents the final drill for dense-bone.




Tapered Screw-Vent, Trabecular Metal & Eztetic Drilling Sequence Dríva Gold Drills


3.7mmD TSV & TM Implants



3.7 mmD

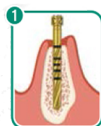


TSV23G
2.3 mmD
Drill

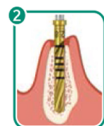

FOR SOFT BONE
TSV28G
2.8 mmD
Drill



FOR DENSE BONE
TSV34D28G
3.4/2.8 mmD
Step Drill

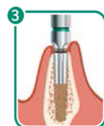


OPTIONAL FOR
DENSE BONE
TT3.7
3.7 mmD
Cortical Bone Tap

Attention: Bone Tap for placement in dense bone; 22mmL drills shown.
16mmL drills have depth markings for placement of 8 and 10mm Implant lengths.


4.1mmD TSV & TM Implants




4.1 mmD

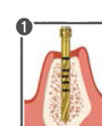

TSV23G
2.3 mmD
Drill

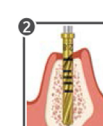


TSV28G
2.8 mmD
Drill

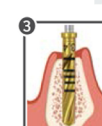


FOR SOFT BONE
TSV34G
3.4 mmD
Drill



FOR DENSE BONE*
TSV38D34G
3.8/3.4 mmD
Step Drill

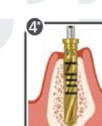

TRABECULAR METAL ONLY
FOR DENSE BONE*
TSV38G
4.1 mmD
Cortical Bone Tap

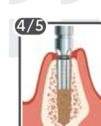


OPTIONAL FOR
DENSE BONE
TT4.1
4.1 mmD
Cortical Bone Tap

*When placing the 4.1mmD Trabecular Metal Implant in dense bone (type D1), add an additional step utilizing the TSV38G drill after TSV38D4G.


4.7mmD TSV & TM Implants



4.7 mmD

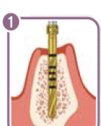


TSV23G
2.3 mmD
Drill

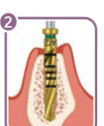


TSV34D28G
3.4/2.8 mmD
Step Drill



FOR SOFT BONE
TSV38G
3.8 mmD
Drill

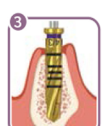


FOR DENSE BONE
TSV44D38G
4.4/3.8 mmD
Step Drill

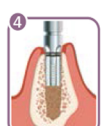


OPTIONAL FOR
DENSE BONE
TT4.7
4.7 mmD
Cortical Bone Tap




6.0 mmD

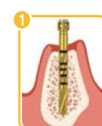

TSV23G
2.3 mmD
Drill

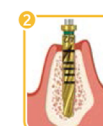

TSV34D28G
3.4/2.8 mmD
Step Drill

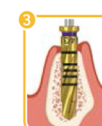


TSV44D38G
4.4/3.8 mmD
Step Drill



FOR SOFT BONE
TSV51G
5.1 mmD
Drill



FOR DENSE BONE*
TSV57D51G
5.7/5.1 mmD
Step Drill

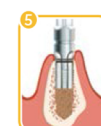

OPTIONAL FOR
DENSE BONE
TT6.0
6.0 mmD
Cortical Bone Tap

*In dense bone, an optional additional step drill may be used before TSV57D51G > TSV51D44G.
Note: this additional drill is sold separately and is not included in kits.


3.1mmD Eztetic Implant




3.1 mmD



O201G
2.1/1.6 mmD,
8 mmL - 11.5 mmL
Drill

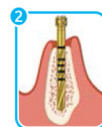


TSV23G
2.3 mmD
Drill

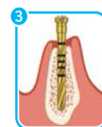

EZT24G
2.4 mmD
Drill



EZT28D24G
2.8/2.4 mmD
Step Drill

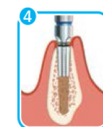

Optional for
Dense Bone
ZOPT30
3.0 mmD
Cortical Bone Tap